



Break glass access

Break glass access provides clinicians with self-service access to patient studies completed at a Light Radiology clinic where you were not the referring doctor.

1. **Open InteleViewer and login**
2. **Click the Padlock Break glass icon**

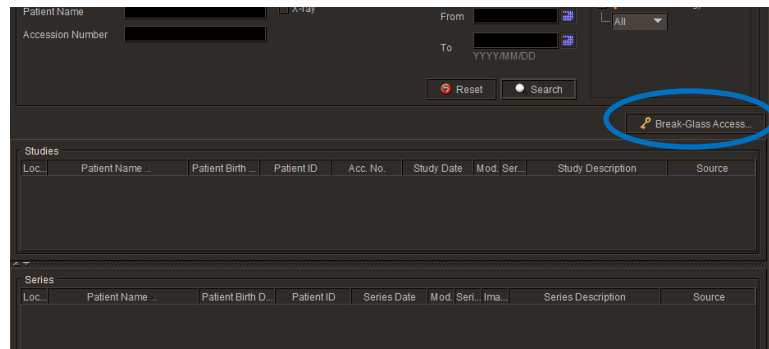

3. **Read and accept the confidentiality agreement by ticking the acknowledgment box**

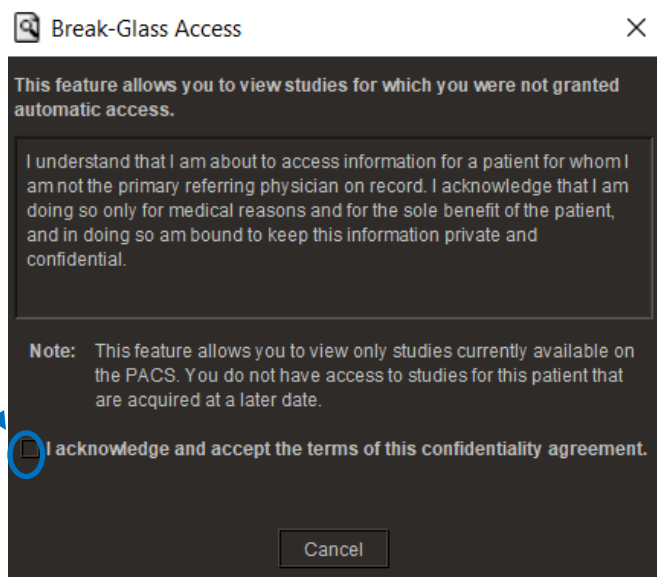

4. Search for the patient required

Break-Glass Access

This feature allows you to view studies for which you were not granted automatic access.

I understand that I am about to access information for a patient for whom I am not the primary referring physician on record. I acknowledge that I am doing so only for medical reasons and for the sole benefit of the patient, and in doing so am bound to keep this information private and confidential.

Note: This feature allows you to view only studies currently available on the PACS. You do not have access to studies for this patient that are acquired at a later date.

I acknowledge and accept the terms of this confidentiality agreement.

Search patients by Patient ID Patient name AND date of birth

Test, Patient AND 19800607
Last, First, Middle ? YYYYMMDD

Search

Cancel

5. Click Access Patient Studies

Search

A patient that matches your search criteria was found:

K12345 - Sample, Patient (M 43Y)

Note: Please allow a few minutes for the studies to become available.

Access Patient Studies Cancel

6. The study will appear in the search window, double click on the study to open.